



Intraosseous Infusion System

EZ-Stabilizer

The EZ-Stabilizer® is a strong medical adhesive tape that is simple to apply and provides good skin adhesion for secure placement of the EZ-IO Needle Set. Designed for 15mm, 25mm, and 45mm EZ-IO catheters regardless of location.

Features and Benefits:

- Hypoallergenic Pressure Sensitive Adhesive
- Translucent
- No Natural Rubber Latex
- Waterproof Barrier
- Flexible & Printable



Composition:

- Tape Caliper—Polyethylene Film Tape
- Backing—Translucent Polyethylene
- Adhesive—Acrylate, designed for medical/surgical use

Packaging and Recommended Storage:

The EZ-Stabilizer has a four year expiration date.

Ordering Information:

Part #	Description	Contents
9066	EZ-Stabilizer	1 EZ-Stabilizer



1-866-479-8500

Vidacare.com


M-211 Rev C

Immediate Vascular Access...
When You Need It.SM

EZ-STABILIZER


DIRECTIONS FOR USE

Placement

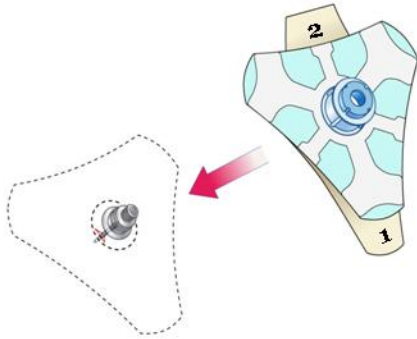


1.

- A. Centrally locate catheter to the eye of the stabilizer
- B. Place stabilizer on catheter

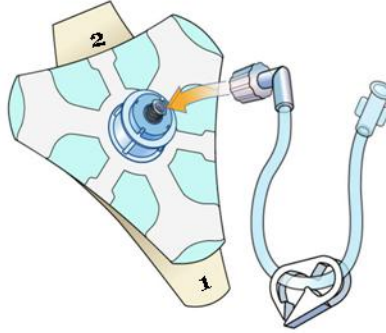

2.


- A. Attach EZ-Connect to stabilizer

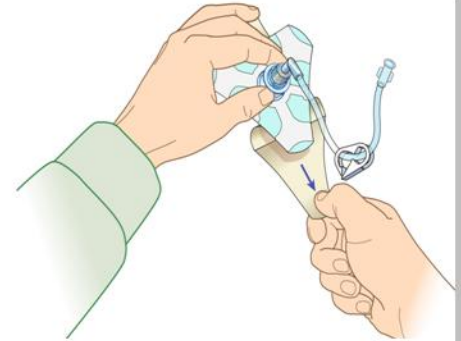

3.


- A. Remove stabilizer's adhesive backing
- B. Grasp stabilizer and gently pull tabs 1 and 2

Removal

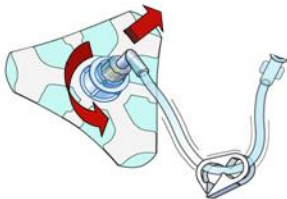


1.

- A. Remove EZ-Connect

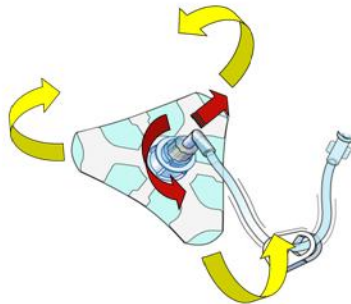

2.

- A. Lift adhesive dressing

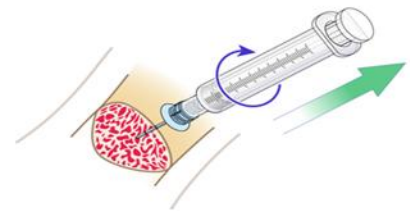

3.

- A. Remove catheter