
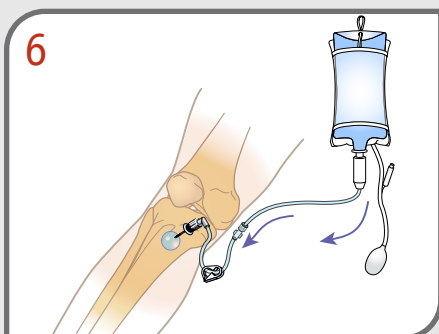
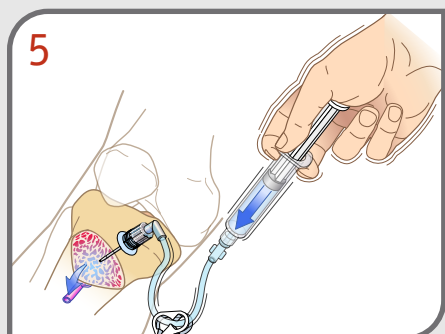
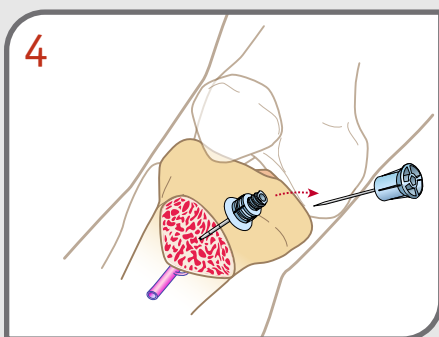
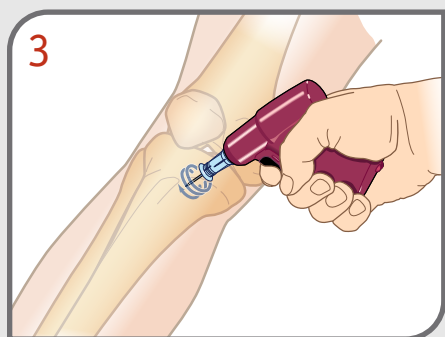
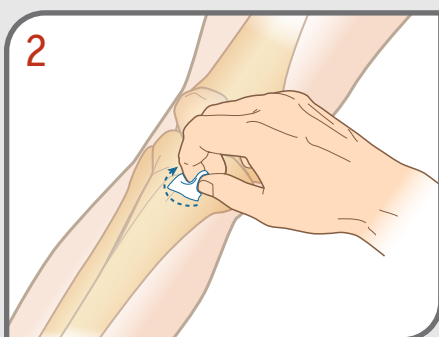
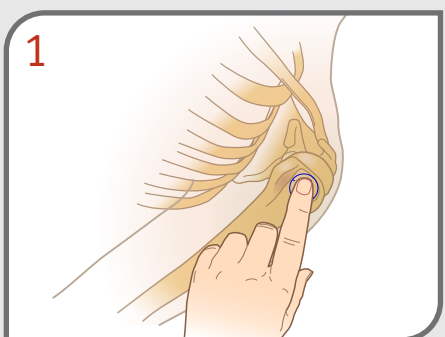
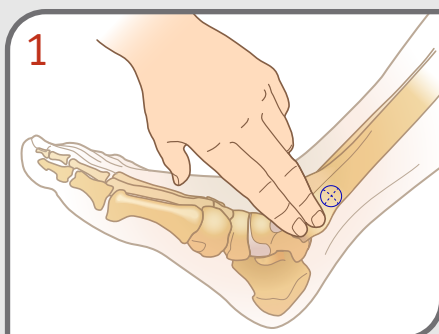


Intraosseous Infusion System

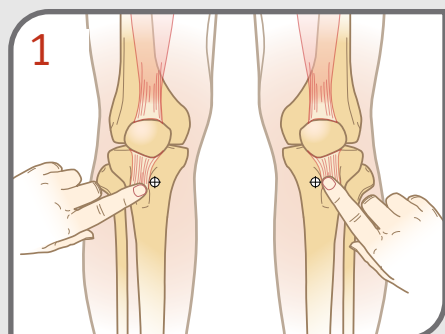

To Insert Needle Set:


- Locate landmarks 1
- Clean site 2
- Insert EZ-IO[®] Needle Set 3
- Remove stylet from catheter 4
- Attach primed EZ-Connect[®]
- Consider IO 2% lidocaine without preservatives or epinephrine (cardiac lidocaine) for patients responsive to pain – prior to flush

Follow institutional protocols/policy

- Medications intended to remain in the medullary space, such as a local anesthetic, must be administered very slowly until the desired anesthetic effect is achieved
- Syringe bolus (flush) IO with 10 ml normal saline..... 5
- Start infusion under pressure 6

A Medical Director or qualified prescriber must authorize appropriate dosage range.

Do Not Leave the EZ-IO catheter in for more than 24 hours.

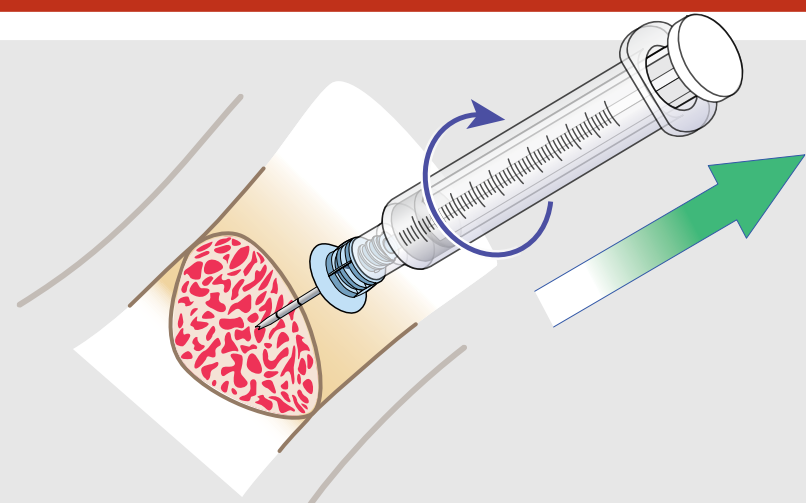

To Remove Catheter:

- Stabilize patient's extremity
- Connect sterile Luer lock syringe to hub of catheter
- Rotate catheter clockwise – while pulling straight back
- When catheter has been removed, immediately place in appropriate biohazard container.

DO NOT ROCK the catheter while removing. Rocking or bending the catheter may cause the catheter to separate from the hub.

Emergency contact in US or Canada call: 1-800-680-4911

For international assistance contact your local Vidacare Distributor