
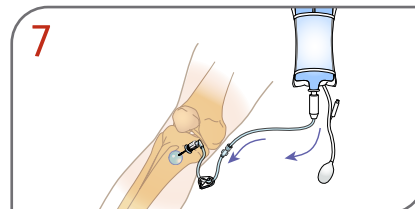
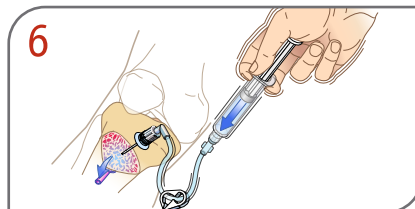
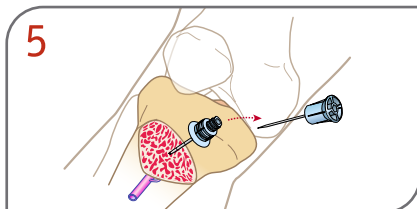
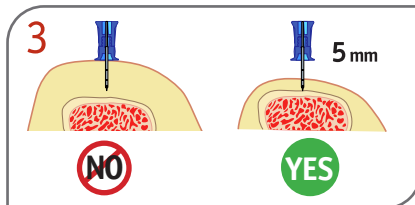
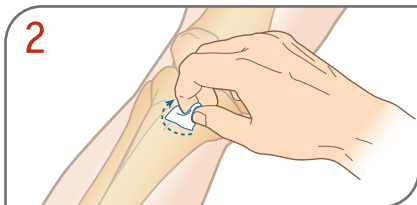
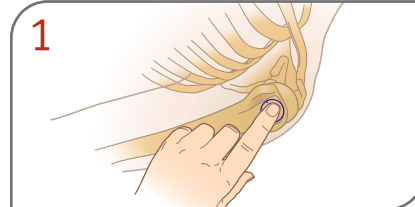
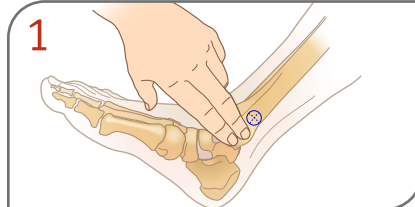


## Intraosseous Infusion System

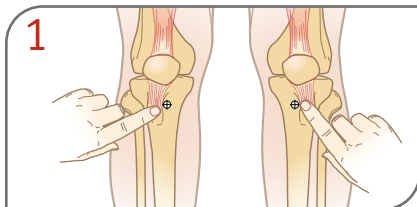

## Intraosseous Infusion System (continued)

### Insertion Instructions:

- 1 Protect yourself (BSI/PPE)
- 2 Identify indication & appropriate site
- 3 Check for contraindication
- 4 Locate landmarks ..... 1
- 5 Clean site ..... 2
- 6 Prepare driver and Needle Set
- 7 Insert Needle Set, confirm black line /5 mm mark ..... 3
- 8 Gently complete insertion ..... 4
- 9 Stabilize Needle Set & remove driver
- 10 Remove stylet and place in an approved sharps container ..... 5

- 11 Confirm placement
- 12 Attach primed EZ-Connect® extension set
  - Consider IO 2% lidocaine without preservatives or epinephrine (cardiac lidocaine) for patients responsive to pain – prior to flush


#### *Follow institutional protocols/policy*

- Medications intended to remain in the medullary space, such as a local anesthetic, must be administered very slowly until the desired anesthetic effect is achieved
- 13 **Syringe FLUSH IO** with 10 ml of a sterile solution ..... 6
  - 14 Start infusion (use pressure) ..... 7
  - 15 Stabilize catheter as needed

## Remove within 24 hours

### Removal Instructions:

- 1 Protect yourself (BSI/PPE)
- 2 Ensure alternate access
- 3 Disconnect EZ-Connect®
- 4 Attach sterile (zeroed) Luer lock syringe
- 5 **Continuously rotate** clockwise **while pulling**
- 6 Dress site & monitor for complications

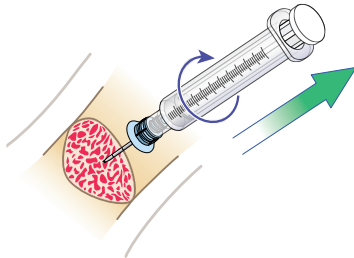

©Vidacare®2010 P/NVID05-9009 REV. E

Clinical Support **1.800.680.4911**

Vidacare.com